ThevoChair

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GB

Locking Mechanism for Rocking Motion

Item code: 63916

Carefully tip the ThevoChair on the side and lay it on a clean surface. The locking mechanism has to be installed at the marked position between both metal tubes under the seating surface. (Fig. 1)

Please remove bolt from part B of the locking mechanisms (Fig. 2).

Insert part A as shown from the front side of the product between the tubes. Make sure that the lever is pointed to the back of the product. The metal bracket should be positioned above the middle metal base. (Fig. 3)

Afterwards please screw part B and A together and make sure that the tubes of the ThevoChair are located in the provided indentation of the metal bracket. Please tighten the screw and secure. (Fig. 4)

Carefully turn up the ThevoChair again. Please use locking mechanism from the back side of the product. To lock out the rocking movement please turn the lever to the top (Fig. 5), for release turn to the left side (Fig. 6).

Warning: Do not insert your fingers into the locking mechanism. Please do not use the locking mechanism while the chair is rocking.

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